EQUITAMP®

**EQUITAMP
KNITTED®**

**EQUITAMP
FIBRILLAR®**

**EQUITAMP
FIBRILLAR THIN®**

**EQUITAMP FIBRILLAR
1 LAYER®**

**EQUICEL
POWDER®**

**EQUICEL
GEL®**

**ABSORBABLE HEMOSTATS
HÉMOSTATIQUE RÉSORBABLE
HEMOSTÁTICO ABSORBIBLE**

CE 0373

#madeinitaly

EQUIMEDICAL®

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Main characteristics of Equitamp®:

- Rapid hemostatic effect (2 - 4 minutes)
- Completely absorbable
- No harmful tissue reaction or reaction of hypersensitivity
- Maximum resistance and manageability
- Bactericidal effect
- Absorption times (4-6 days)

Indications:

Equitamp® is used adjunctively in surgical procedures to assist in the control of capillary, venous, and small artery haemorrhages when ligation or other conventional methods of control are impracticable or ineffective.

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The "Knitted" version is three times more thick and is characterized by a unique manufacturing process that offer maximum resistance to tearing. The product can be cut easily and applied according to the various surgical requirements.

Main characteristics of Equitamp Knitted®:

- Rapid hemostatic effect (2 - 4 minutes)
- Completely absorbable
- No harmful tissue reaction or reaction of hypersensitivity
- Easy to cut and to conform
- Maximum resistance and manageability
- Bactericidal effect

Indications:

Equitamp Knitted® can be used to control major bleeding such as par-enchymatous organs.

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1 Layer

Thin

Absorbable hemostat of oxidized regenerated cellulose with Ph acid

Main characteristics of Equitamp Fibrillar®:

- Rapid hemostatic effect (2-4 minutes)
- Made in fibrillar form
- Completely absorbable
- No harmful tissue reaction or reaction of hypersensitivity
- Maximum resistance and manageability
- Bactericidal effect

Indications:

Equitamp Fibrillar® is used adjunctively in surgical procedures to assist in the control of capillary, venous, and small artery haemorrhages when ligation or other conventional methods of control are impracticable or ineffective. The unique softness makes it particularly indicated for laparoscopic use.

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Absorbable hemostat of oxidized regenerated cellulose with neutral Ph, in powder shape, sterile, ready to use.

Main characteristics of Equicel Powder®:

- Rapid hemostatic effect (2-4 minutes)
- Bactericidal effect
- Completely absorbable through hydrolysis (5-8 days) also in blood absence
- Due to its neutral PH, does not inactivate thrombin
- No harmful tissue reaction or reaction of hypersensitivity

Indications:

Equicel Powder® doesn't need to be assembled, must be used dry and directly sprayed where necessary the haemostasis. His exclusive sterilized dispenser allows an easier and faster application even in difficult surgical procedures when the area is hard to reach. Equicel Powder® can be used in many areas of surgery: e.g. plastic surgery, gynaecology, gastric resection, liver and gall bladder operations, thoracic and abdominal sympathectomy, thyroid operations, skin transplant, treatment of superficial injuries.

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Absorbable hemostat of oxidized regenerated cellulose with neutral Ph, in gel, ready to use.

Main characteristics of Equicel Gel®:

- Rapid hemostatic effect (2-4minutes)
- Bactericidal effect
- Completely absorbable through hydrolysis also in blood absence
- Due to its neutral PH, does not inactivate thrombin
- No harmful tissue reaction or reaction of hypersensitivity

Indications:

Equicel Gel® can be used in different gynaecological and cholecyst. Indicated for open and laparoscopic surgery.

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